

sonataCare

HOSPITAL TRACKING SYSTEM




Product Outline

Screen Tracks

The System 90 Meditrack Bedscreen and Mini Track systems are ideal for use in, but not limited to, intensive care units, emergency departments, nursing homes, aged care, general practices and dental surgeries to help facilitate patient privacy. Extruded from commercial grade aluminium the Bedscreen track is strong and versatile and can reach spans of up to 3 metres long without supports. If distances are greater than three metres, or supports are required, the track can be suspended using dropper rods, top fixed or fixed using wall brackets (please refer to the installation page for more details)



Track HT6000

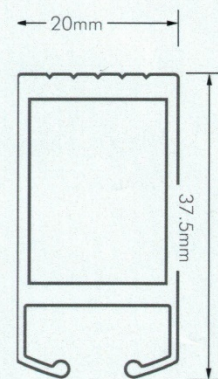


Track HT6000 Profile

Mini Track is ideal for the use of shower cubicle tracking carrying lightweight shower screens





Track MT6000

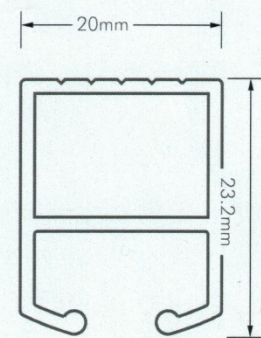

Track MT6000 Profile

Available in both clear anodised and powder coated white, the Bedscreen and Mini Track can provide you with a cost effective means to provide privacy to any area. The composition of the aluminium also allows both the tracks to be bent to specification.

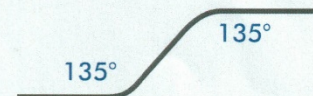
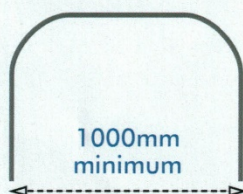


Product Outline



Cassette System

The curtain cassette system allows for easy removal of the curtains without lifting and replacing the curtain hook by hook. With this system you are able to draw the curtain into the cassette and with a release of the knob the entire curtain will fall away for easy maintenance. For your convenience the cassettes are available in left, right and top release functions.

SIDE FIX

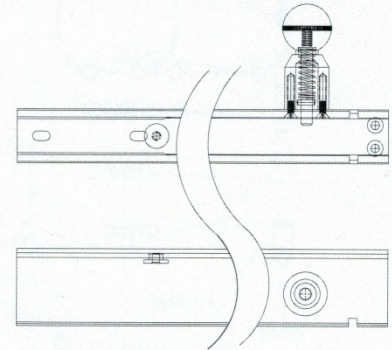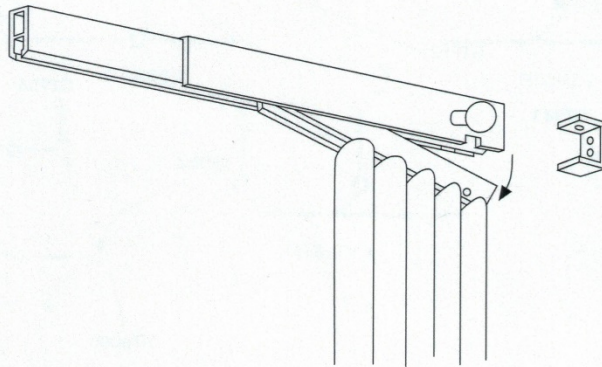

TOP FIX

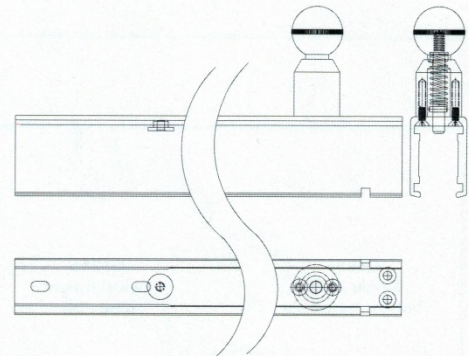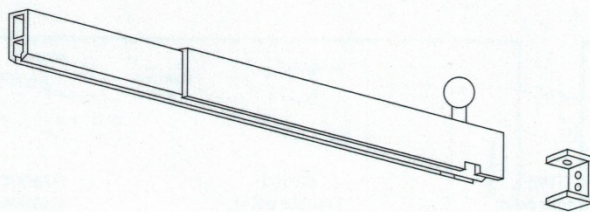


How to load the Cassette System


- 1 With the curtain attached slide the rotatable runners (GT931) into the cassette 'inner'.
- 2 To lock the cassette 'inner' into the outer casing of the cassette (which has already been fitted) slot the 'inner' against the stopper at an angle and bring the other side up. Release the knob and bring the 'inner' into the outer casing, making sure the knob retracts back into the cassette system.
- 3 You can now draw the curtain out of the cassette and along the bedscreen track. To remove the curtain, reverse the process.




Product Applications

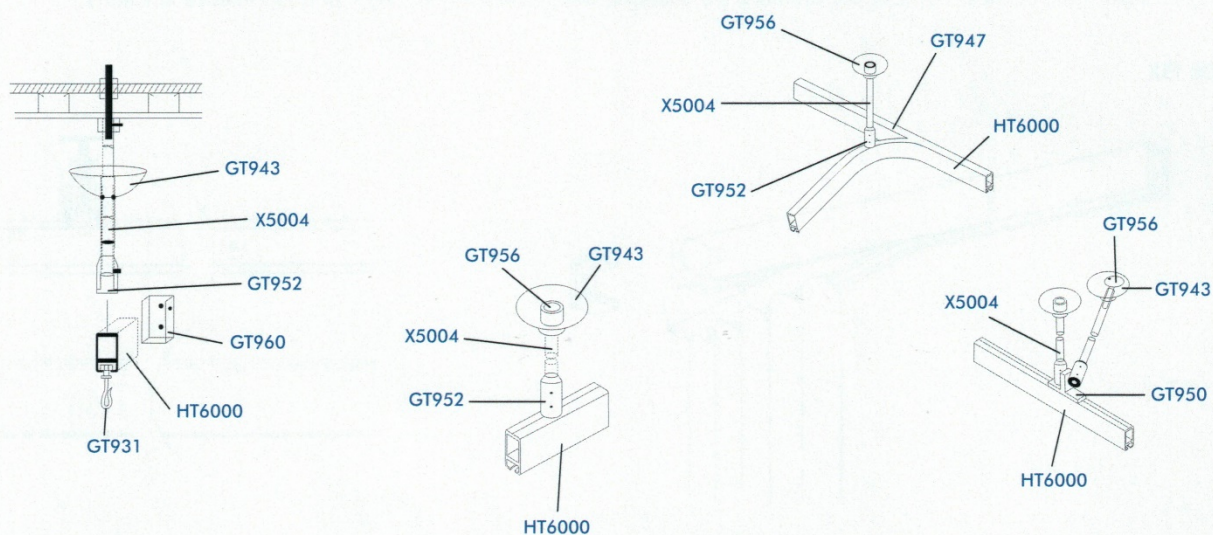


GT951
Single point
hanger assembly

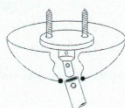


GT956
Swivel mount
assembly

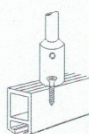


GT951
Single point
hanger

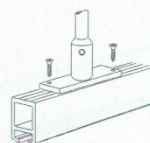


GT952
Double point
hanger assembly

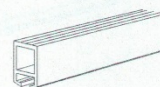


HT6000
MT6000

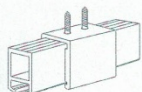


GT953
Ceiling
mount clip

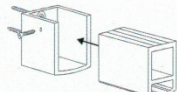


GT960
Wall mount
socket

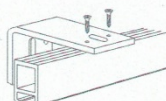


GT955
Steel wall
bracket

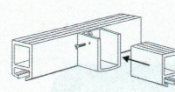


GT960
Wall mount
socket

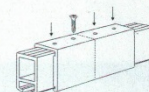

GT947
Joining splice

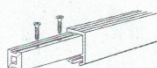


GT946
Internal
track joiner

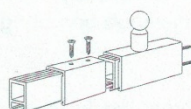

GT949
Cassette
loader

Typical Room Designs

System 90 Meditrack

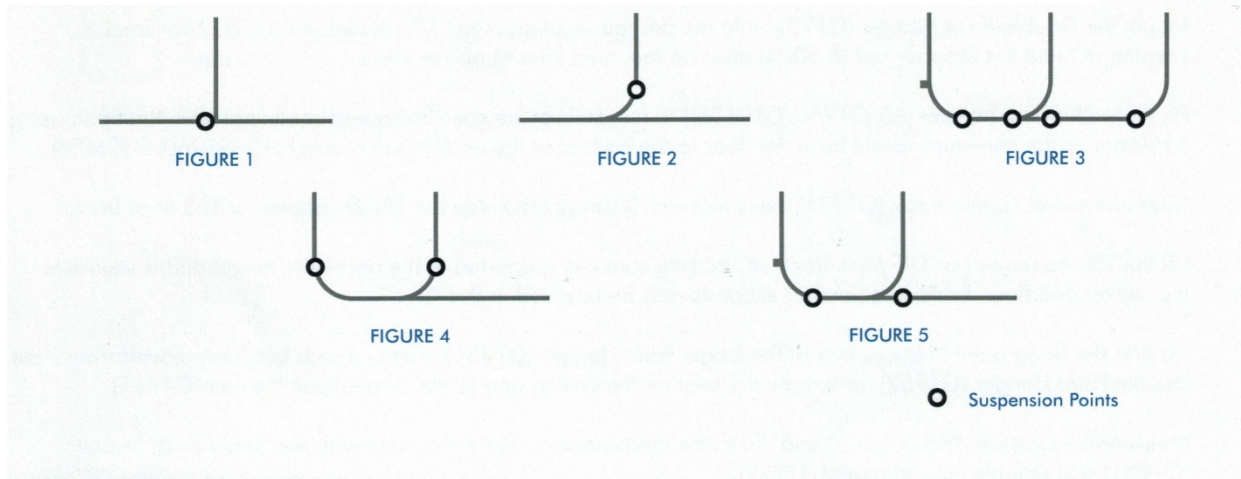

System 90 Mini Track

